


Tamarack Clevisphere™ Joint

Range of Motion

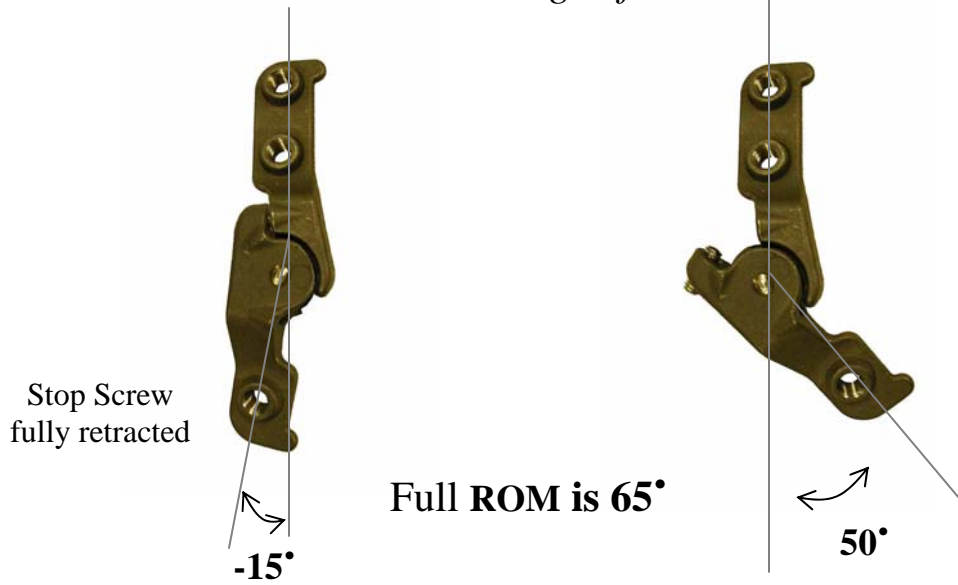

ROM with adjustable stop

With 10mm screw:




With 14mm screw:



Stop Screws fully extended

For use as a Dorsiflexion Stop: Non-Ambulatory Only

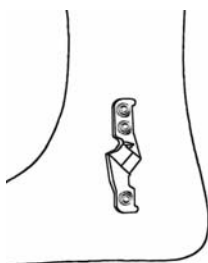

Tamarack Habilitation Technologies, Inc. does not endorse the use of this joint as a variable dorsiflexion stop in an ambulatory orthosis.

In almost all ambulation situations, stopping dorsiflexion subjects the joint and its anchorage to much higher moments than the joint was designed for.

However, there are non-ambulatory variable dorsiflexion stop applications for which this joint is useful.