

Tamarack® Flexure Joint Caps™

Tamarack Flexure Joint Caps™ are a uniquely designed external mounting option for Tamarack Flexure Joint® installation.



Tamarack Flexure Joint Caps provide ideal joint cavities for articulating thermoplastic and carbon fiber devices. They also provide a time & cost-saving alternative when articulating an existing solid ankle foot orthosis.

Each Tamarack Flexure Joint Cap Package Includes:

Installation instructions, (2) pair of Flexure Joint Caps (black or natural color), drill template, mounting hardware, and ShearBan patches for a clean, aesthetic finish. Tamarack Flexure Joints sold separately.

Black: Order Part #741-CAP-LB

Natural: Order Part #741-CAP-L

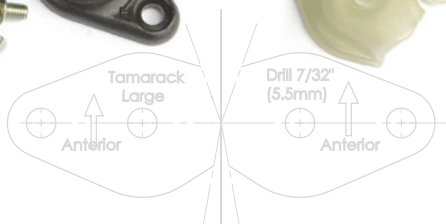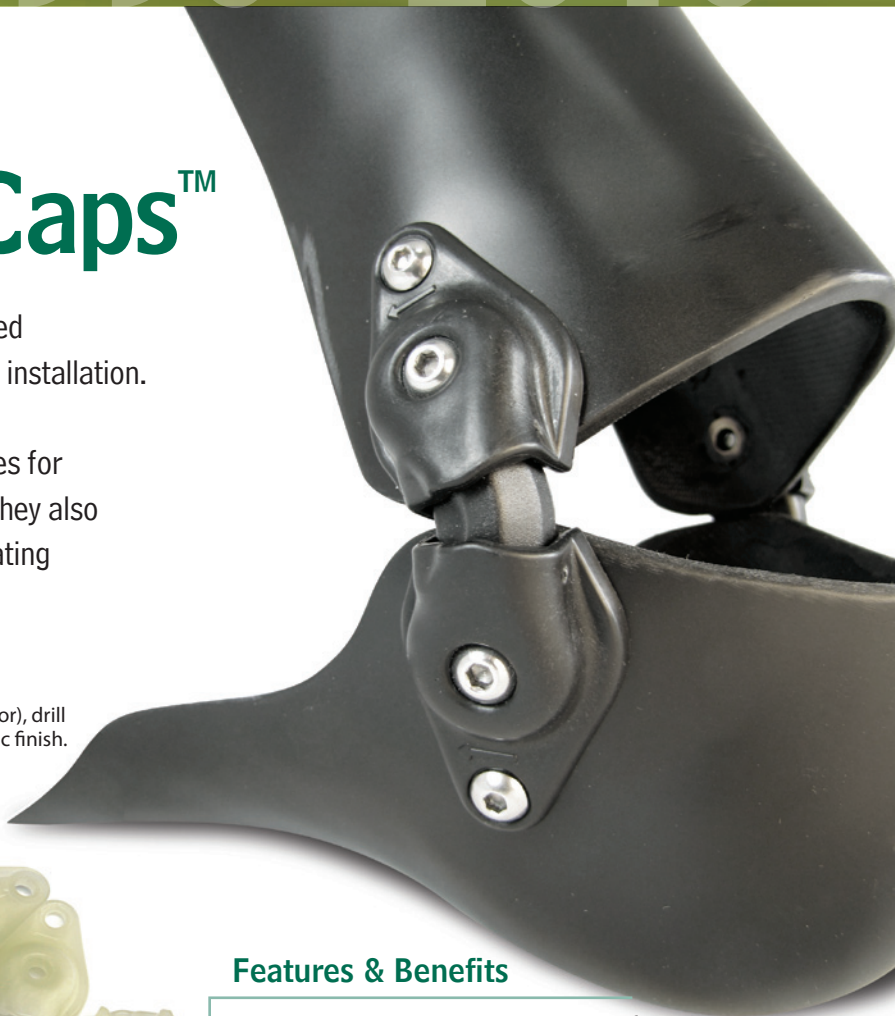

Features & Benefits

- Engineered to produce an intimate fit with Tamarack Flexure Joints
- Eliminates the need to fabricate a new brace when articulation is desired
- Low-profile design
- Available in black or natural color options
- Works well with a variety of bracing materials:
 - thermoplastic
 - copoly
 - carbon fiber
 - metal & leather systems

Installation video and additional product information available at www.tamarackhti.com

How to use Tamarack Flexure Joint Caps

- (1) Use template enclosed with kit to locate attachment holes and section lines.
- (2) Drill holes; then trim foot plate and calf section.
- (3) Attach caps over any model Tamarack Flexure Joint (free motion or dorsiflexion assist).

How to purchase Tamarack Flexure Joint Caps

Contact Becker Orthopedic
Phone (248) 588-7480 | Fax (800) 923-2537
Toll free (800) 521-2192

Or visit BeckerOrthopedic.com
for a list of distributors worldwide