




Application technique for transfer boards with assistive handles:

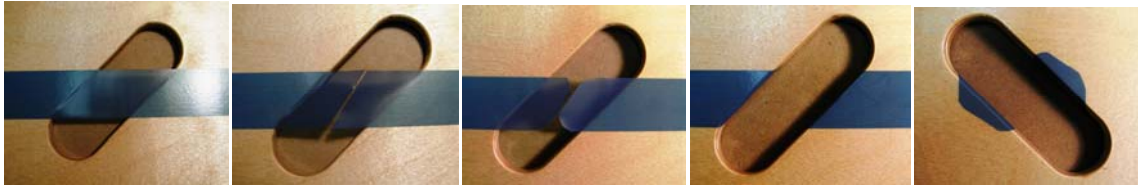

- Step 1: Apply **glidefree™** over top of transfer board handle.
- Step 2: Trim inside the transfer board handle, parallel to the slot.
- Step 3: Round corners, to prevent them from peeling during use.
- Step 4: Wrap strips around back side of transfer board.
- Step 5: Press firmly around ends of **glidefree™** strips to ensure adhesion.

For more information about **glidefree™** transfer board strips, contact
Tamarack™ Habilitation Technologies:

Mail
1670-94th Lane NE
Minneapolis, MN 55449

Phone
763.795.0057 (local)
866.795.0057 (toll-free)

E-mail
info@tamarackhti.com